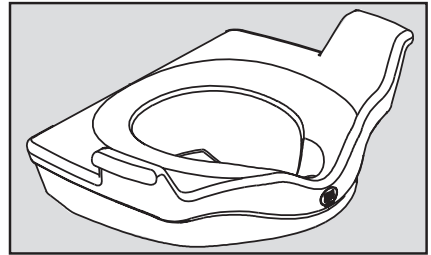
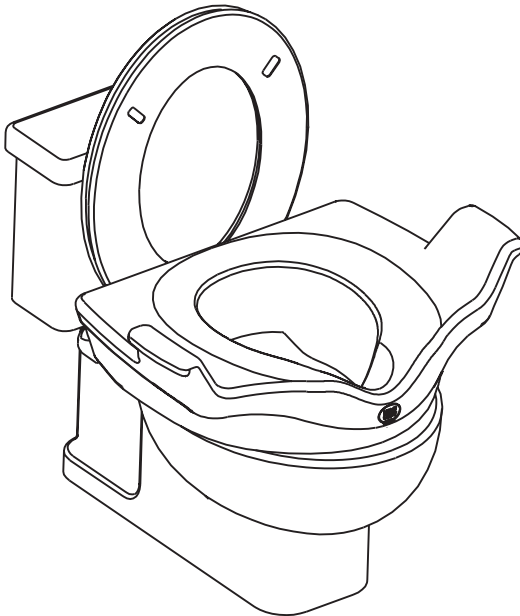
**LOCKING ELEVATED TOILET SEAT WITH SUPPORT HANDLES
INSTRUCTION SHEET**

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Installer:
Leave these instructions with customer

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Don't forget to register and review your product by scanning the code or visiting moen.com/product-registration.

Download reader at getscanlife.com.

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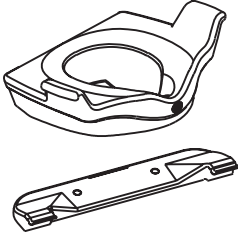
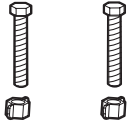
Warning ! – FOLLOW THESE INSTALLATION INSTRUCTIONS CAREFULLY. PROPER INSTALLATION IS THE INSTALLER'S RESPONSIBILITY. FAILURE TO FOLLOW CORRECT INSTALLATION PROCEDURES CAN CAUSE THE SEAT TO BECOME LOOSE, WHICH CAN RESULT IN SERIOUS INJURY.

Specifications:

Fits on most commode configurations – either round or elongated bowl.
Raises seat 5 inches. Toilet seat must be clean and dry before installation.
Plastic rim of seat must fit inside of toilet rim.

Dimensions:

24" wide x 16" deep x 5" high

Installation Instructions:

Tools Required: Standard size screwdriver to attach the bracket.

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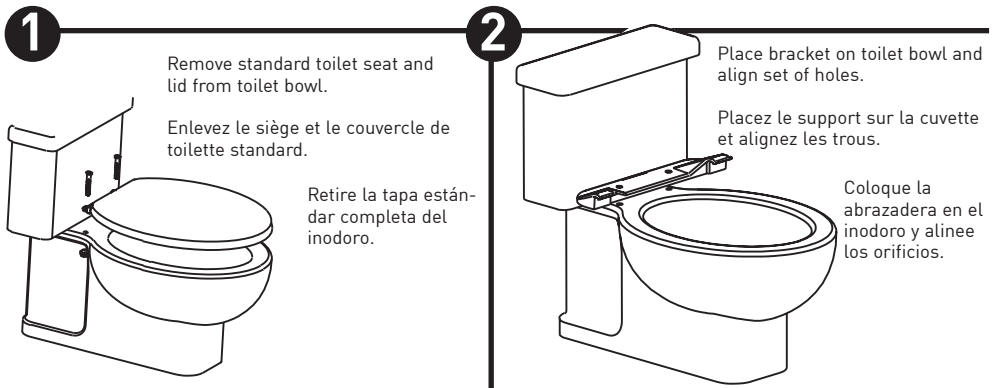
3

Reattach toilet seat and lid with mounting bracket using supplied screws by lining up the holes on the toilet bowl.

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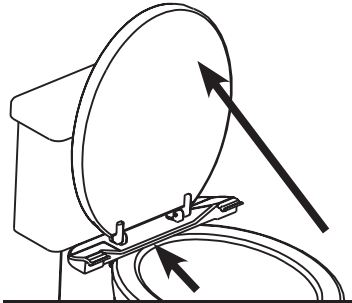
Verify no bracket interference with seat or lid when lowered

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5

Attach bracket tabs in rear slots in elevated seat

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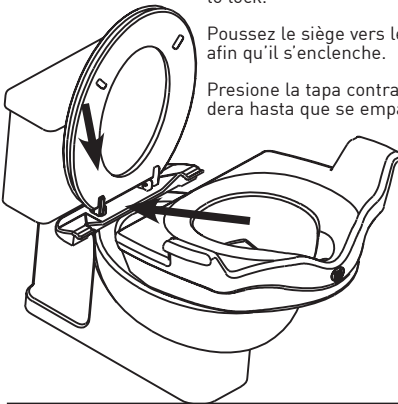
Push the seat towards the bracket to lock.

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
▲ Warning: Fall Hazard! Back of elevated seat must not touch raised toilet seat. DO NOT USE IF BACK OF ELEVATED SEAT CAN TOUCH RAISED TOILET SEAT. Contact with raised toilet seat can force elevated seat off of toilet.

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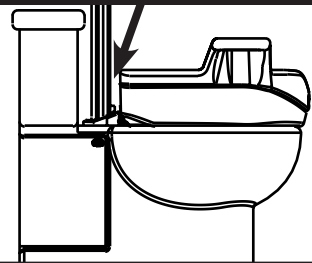To remove, reverse process, lift front of seat and pull.

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WARNING: TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 300 POUNDS. CONSULT HEALTHCARE PROFESSIONAL FOR ADVICE AND INSTRUCTIONS SPECIFIC TO YOUR NEEDS BEFORE USE. PLASTIC INNER RIM OF SEAT MUST FIT INSIDE OF PORCELAIN TOILET BOWL RIM. IF SEAT CAN ROCK FROM SIDE TO SIDE, DO NOT USE. DO NOT TIP SEAT. SIT DOWN ON AND LEAVE SEAT FROM FRONT, NOT FROM SIDE. WHEN SEATED, DO NOT LEAN TO EITHER SIDE OF SEAT WITHOUT PROPER BALANCING. DO NOT JUMP OR STAND ON SEAT. CONTACT CSI IMMEDIATELY IF ANY COMPONENTS ARE MISSING OR DAMAGED OR FOR ASSISTANCE WITH INSTALLATION.

WARNING. FALL HAZARD. SECURELY ATTACH BOTH BRACKET TABS TO BACK OF ELEVATED SEAT. DO NOT USE IF BACK OF ELEVATED SEAT CAN TOUCH RAISED TOILET SEAT. THIS CONTACT CAN FORCE ELEVATED SEAT OFF TOILET.

Limited Lifetime Warranty: If this product fails due to a defect in materials or workmanship at any time during the life of the product, CSI will replace it free of charge, postage-paid. Simply contact CSI customer service at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OF RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

 For additional warranty information, visit www.moen.com/homecare
The Home Care Locking Elevated Toilet Seat meets or exceeds the applicable requirements of the following standards or guidelines:
The Americans with Disabilities Act (ADA)

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