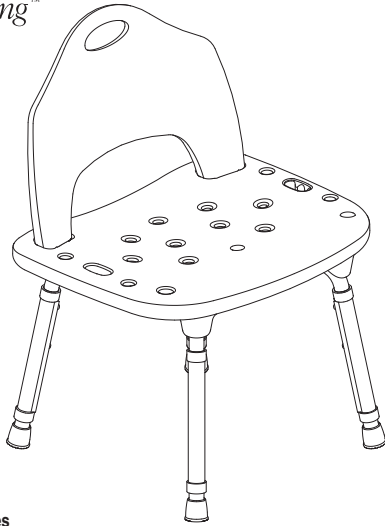
DN7060 HOME CARE® BY MOEN® TOOL-FREE SHOWER CHAIR INSTRUCTIONS

INSTRUCCIONES DE LA SILLA PARA REGADERA SIN HERRAMIENTAS HOME CARE® DE MOEN®
HOME CARE® DE MOEN® - DIRECTIVES POUR L'INSTALLATION DE LA CHAISE DE DOUCHE SANS OUTILS

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WARNING: PLEASE FOLLOW THESE ASSEMBLY INSTRUCTIONS CAREFULLY. ASSEMBLER ASSUMES RESPONSIBILITY FOR PROPER ASSEMBLY. FAILURE TO FOLLOW CORRECT ASSEMBLY PROCEDURES CAN RESULT IN SERIOUS INJURY.

- 1.) Slide adjustable legs onto the stems under the seat. Make sure that the single hole in the top of adjustable leg aligns with the rounded brass stud. Depress brass stud to allow adjustable leg to slide over it, locking it in place (see Figure 1).

To adjust length of chair legs/height of seat (see Figure 2).

FOR MAXIMUM STABILITY, ALL FOUR CHAIR LEGS MUST BE ADJUSTED TO THE SAME LENGTH.

- 2.) Test each leg to make sure that all studs have fully clicked into place.
- 3.) Grasp seat back by handle and insert the tabs into large holes in seat bottom until they snap into place (see Figure 3).

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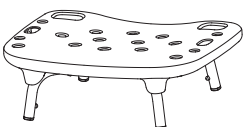
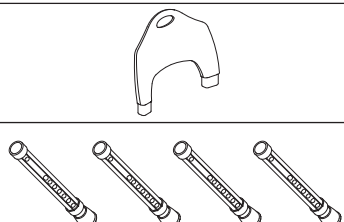
Components Included/Componentes incluidos/Pièces incluses

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Thank you for purchasing this Home Care® by Moen® product. The entire Home Care by Moen family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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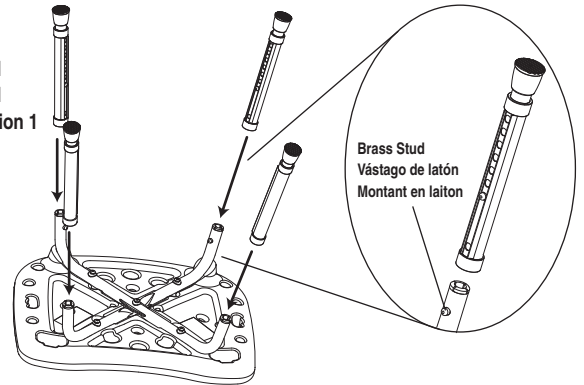
Figure 2
Figura 2
Illustration 2

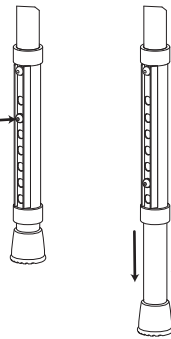
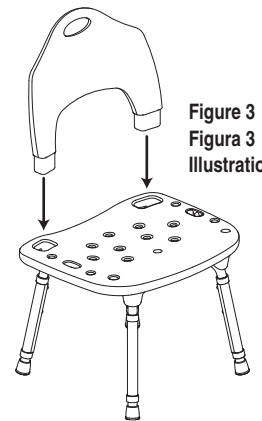
Figure 3
Figura 3
Illustration 3

INNOVATIVE DESIGNS
for independent living™

WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 300 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-882-0116.

LIMITED LIFETIME WARRANTY:

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.homecare.moen.com

Home Care by Moen Tool-Free Shower Chair meets or exceeds the applicable requirements of the following standards/guidelines:

The Americans with Disabilities Act (ADA)

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Creative Specialties is a division of Moen Incorporated. © 2005
Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated.
Home Care and Creative Specialties are a registered trademarks of Moen Incorporated.